

CALF CUFF APPLICATION

Wrap the cuff around the calf and secure the Velcro to hold it in place. Make sure the wrap is snug, but not too tight.

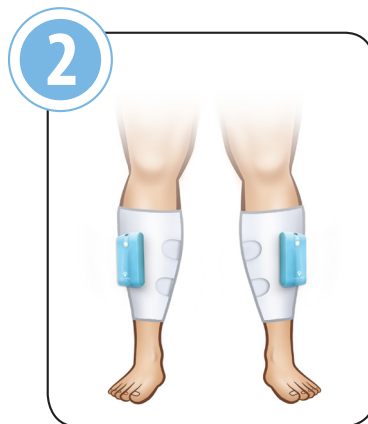
When both wraps are secured on your legs, they should look like the picture above.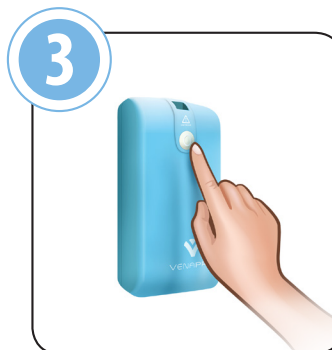

TURNING THE DEVICE ON

When the wraps are secured on your legs, PRESS and HOLD the WHITE power button for about a second until the light is illuminated on each unit.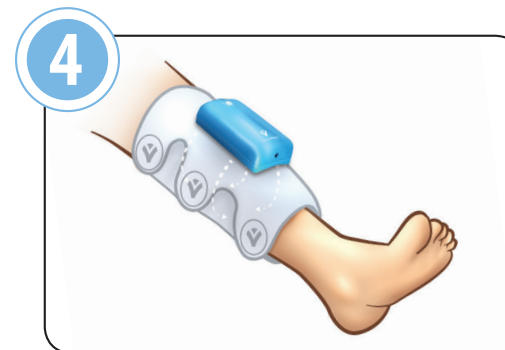

USING THE DEVICE

Don't be startled. The device will make a "humming" sound when inflating and squeezing your leg. **THIS IS NORMAL.** The wraps will inflate once a minute. IF YOU FEEL air releasing around the legs, this is normal. This is a function of the device to keep your legs cool.

PUMP ALARMS

What to do if the Pump Alarms

Don't worry, the alarms are there to insure the units are operating correctly.

BATTERY CRITICAL

If battery voltage drops below a critical level, cycling stops, audible alarm will sound, and UNIT WILL FLASH YELLOW. The Alarm will continue for 30 seconds (unless the unit is powered off) and automatically turn the unit OFF.

LOW PRESSURE

LOW PRESSURE OR LEAK

If the ALARM sounds and the BLUE LED is flashing, the pressure is too LOW. Make sure wrap is attached snugly to the leg. Turn the Unit OFF and then back ON to reset the alarm.

If the unit continues to ALARM after this step, DO NOT try to fix the device. Call Customer Service for a replacement unit at 877-475-7050.

POWER

If the power button illuminates GREEN the units are fully operational.

If the devices DO NOT turn ON, or the power button is "YELLOW" or "FLASHING YELLOW" plug in the power cord immediately to the electrical outlet.

If both units are plugged in, AND turned ON the power button will be GREEN and the batteries WILL charge at the same time.

When the unit is turned OFF and plugged in the battery is charging the POWER BUTTON FLASHES RED.


Once the battery reaches full charge the POWER BUTTON WILL BE RED and REMAIN SOLID.

PLEASE DO NOT CONTACT YOUR PHYSICIAN OR SURGICAL FACILITY FOR ANY QUESTIONS REGARDING THIS DEVICE.

CALL Customer Service: 877-475-7050

PLEASE VISIT: www.InnovaMedHealth.com and click on the PATIENT TAB for an instructional video on the use of this device.

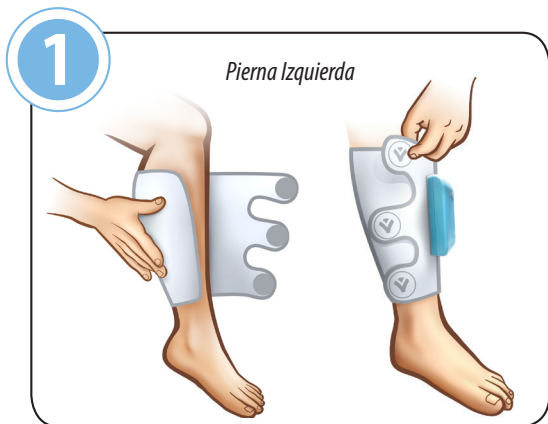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